


# VistaPure™

## BottleFill Remote Water Dispenser

### CVS3050 (S3050) Instructions for Use

The VistaPure™ BottleFill Remote Water Dispenser is an accessory to the VistaPure™ Water Purification System designed to enable filling of independent dental bottles in any location convenient for staff. The dispenser is designed to be installed on the underside of an upper cabinet in the sterilization center.

**Note:** Even if this dispenser is to be used for bottle filling, it is still recommended to install the chrome faucet included with the system at a sink in the steriocenter, as the faucet is the easiest way to drain the water-storage tank when necessary for routine system maintenance.

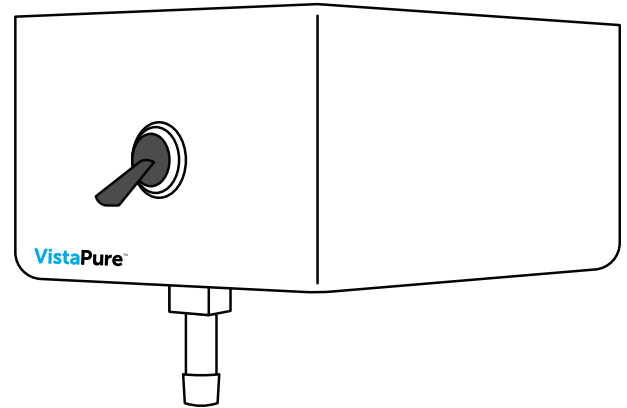

#### ✓ INCLUDED

- Dispenser housing assembly
- 3/8" tee union
- 3/8" O.D. silicon tubing (6")
- Installation instructions


#### ✗ NOT INCLUDED

- Mounting screws  
*Since cabinet substrate unknown*
- 3/8" O.D. LLDPE tubing  
*Since distance to faucet line unknown*

## INSTALLATION INSTRUCTIONS

See **FIGURE A** for typical installation of a single VistaPure™ BottleFill Remote Water Dispenser.

1. Position the dispenser housing in the desired location (usually the underside of an upper wall cabinet), mark the four keyed holes, and drill pilot holes (if necessary).
2. Install the dispenser housing assembly securely by using screws or bolts appropriate for the cabinet substrate and of an appropriate length (not included).
3. Turn off the water inlet supply valve to the VistaPure™ Water Purification System.
4. Turn off the valve on the water-storage tank.
5. Relieve the water pressure from the system by lifting up on the black handle located on the chrome faucet until water slows to a stop. Then close the faucet.

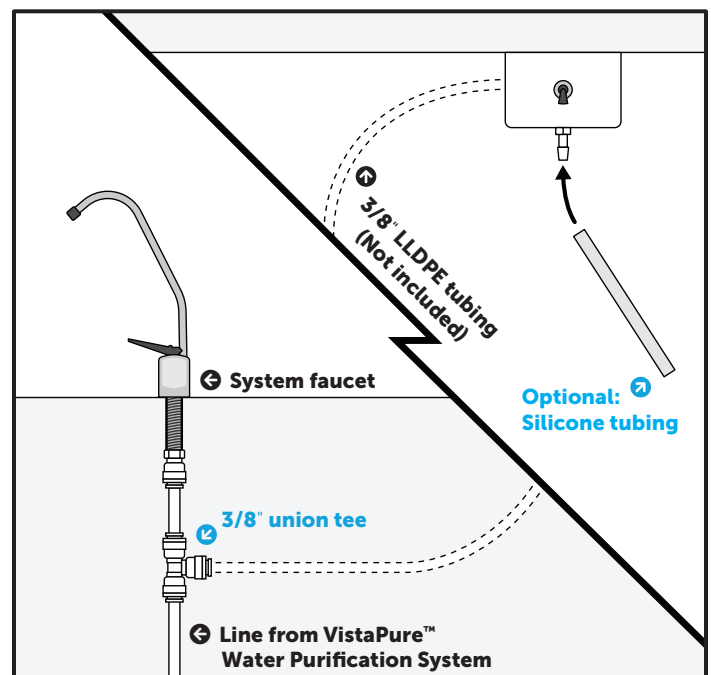

**FIGURE A**

## INSTALLATION INSTRUCTIONS (CONT.)

6. Install the 3/8" tee union in the 3/8" white tubing that runs from the system manifold to the chrome faucet (see illustration). Simply cut the line with a tubing cutter or sharp blade and install the tee by firmly pushing the 3/8" tubing into the quick-connect fittings.
7. Connect a length of white 3/8" O.D. LLDPE tubing (not included) to the tee and run the tubing to the dispenser housing assembly. Typically, this 3/8" line runs to the back of a base cabinet and up through a chase or wall to the wall-cabinet location.
8. Connect the 3/8" O.D. LLDPE tubing to the included swivel elbow fitting on the dispenser housing. The chosen location will determine the orientation of this swivel fitting. Make sure the tubing is fully inserted into the fitting to avoid leaks.
9. When all connections have been made, slowly open the water inlet valve on the system, open the tank valve and make sure the chrome faucet handle is in the OFF position. Wait for about five (5) minutes and check for leaks anywhere around the tee and dispenser housing assembly.
  - If any leaks are found, make corrections by simply depressurizing the system, relieving water pressure and reseating the tubing in the fittings. Check again for leaks.
10. If no leaks are present, actuate the toggle switch on the front of the dispenser housing assembly and run water into a bottle, cup or bucket to ensure water flow.
11. Sanitize the system storage tank, faucet, and water-dispenser lines following the instructions included in the VistaPure™ Water Purification System manual.

## MULTIPLE-DISPENSER INSTALLATION OPTIONS

If having multiple bottle-filling stations would be convenient, two or more VistaPure™ BottleFill Remote Water Dispensers can be installed, either in parallel (**Figure B**) or series (**Figure C**).

### USING THE VISTAPURE™ BOTTLEFILL REMOTE WATER DISPENSER

After installed and sanitized, simply position the dental bottle beneath the barb, then lift up on the toggle to flow purified water from the VistaPure™ Water Purification System into the bottle. When the bottle is full or the desired amount dispensed, simply release the toggle and water flow will stop.

The included silicon tubing is intended to serve as an optional extension to the dispenser. This can be convenient for smaller bottles set on the countertop directly beneath the dispenser. Simply cut the silicon tubing to the desired length, push the tubing over the barb on the dispenser, and insert the other end of the tubing opening of the dental bottle.

If used, the silicon tubing should be autoclaved and/or cleaned with an appropriate cleanser on a regular basis. Simply pull the tubing off the dispenser barb, clean it, and return it to the barb.

For questions about installation, use or maintenance of the VistaPure™ BottleFill Remote Water Dispenser, visit [vrg.support](http://vrg.support) or call Customer Service at (800) 531-2221.

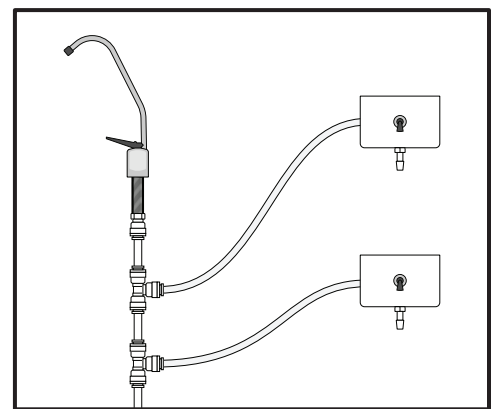


FIGURE B

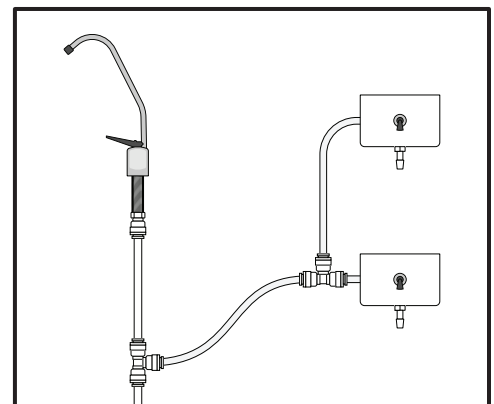



FIGURE C

 Crosstex International, Inc.  
1733 W. Parkside Lane, Phoenix, AZ 85027 USA

Made in USA

Part No. VM1086 / REV A - 04/2020

All product names are trademarks of Crosstex International Inc., a Cantel Medical Company, its affiliates or related companies, unless otherwise noted.

 (888) 276-7783

 [crosstex.com](http://crosstex.com)

**CROSSTEX**  
A CANTEL MEDICAL COMPANY