

SciCan
 www.scican.com



92%
 overall
 rating



This product
 will be available
 in January, 2017.

Description

OPTIM1 is a one-step disinfectant and cleaner wipe with a 1-minute TB kill time and bactericidal claim, and 30-second virucidal and fungicidal claims. It is EPA-registered and the wipes are designed to hold sufficient liquid to meet its contact times. **OPTIM1** contains 0.5% hydrogen peroxide, which breaks down naturally into water and oxygen, in a proprietary formulation and has a natural scent. The manufacturer recommends to rinse all surfaces with soap and water when switching to **OPTIM1** to remove any residual residue from your previous disinfectant. **OPTIM1** is nontoxic and nonirritating to the skin, eyes and respiratory tract. It is listed as a category IV, the EPA's lowest toxicity category requiring no precautionary or hazard warnings. No GHS symbols or communication are required. **OPTIM 1** is also compatible with all clinical contact surfaces, including dental chairs.

Indications

- Use for cleaning and disinfecting of clinical contact surface in accordance with infection control recommendations.

Unique Features

OPTIM1 is a proprietary formulation with a 1-minute kill time that is suitable for use on all clinical contact surfaces and can be safely handled.

Consultants' Comments

- "As safe as it is effective."
- "**OPTIM1** is easy to use, has no aerosol, is easy on sensitive skin, and does not produce eye reactions."
- "Great scent and seemed to clean very well."
- "Did not leave a film on surfaces."
- "Extremely quick action while having broad-spectrum effectiveness."
- "No smell, no residue."
- "Not too wet and the wipes clean really well."
- "Easy to dispense from the container."
- "We like the one-step process, it is easier and the two-step process is too time consuming"
- "We will switch to **OPTIM1** as it is less irritating."

Clinical Tips


- Pull out wipe at a 45 degree angle to avoid pulling out multiple wipes.
- When switching to **OPTIM1** rinse all surfaces with soap and water to remove any residual residue from your previous disinfectant.

Evaluation Highlights

OPTIM1 was evaluated by 37 consultants.

- **OPTIM1** wipes are easy to use, and do not dry out too quickly or too slowly.
- Easy on sensitive skin and did not produce eye reactions.
- Great scent and cleaned very well. Scent was a bit strong.
- Did not leave a film or residue on surfaces, nor did it stain surfaces.
- Quick acting, while having broad-spectrum effectiveness.
- Excellent kill times.

Key Features:

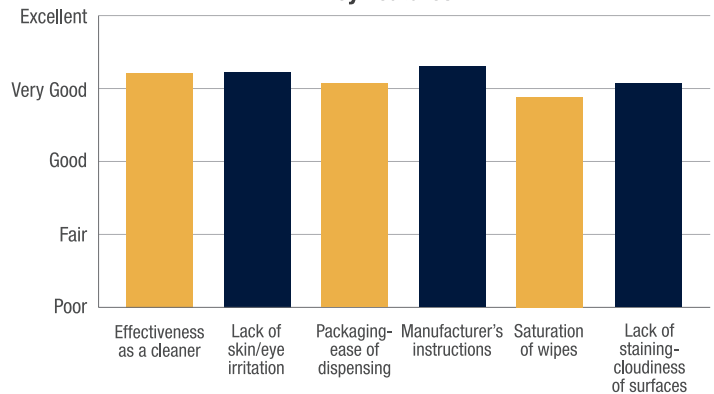

Percentage of Consultants Who Would:

